PFLEGEANLEITUNG BEZUGSMATERIALIEN. CARE AND CLEANING GUIDE COVER FABRICS.

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Thank you for choosing a GREINER brand product – congratulations to a top-class choice – particularly when it comes to upholstery and fabrics. There are two types of GREINER cover fabric to choose from: the diverse „compact.“ range, and the innovative „premium.“ range, which is setting new standards with its ultra-smooth surface and high abrasion resistance.

Each type of cover fabric promises to stand for long-lasting quality, if it is disinfected and cleaned in a regularly and durable manner. Quick-and-easy hygiene measures help to fulfill the daily care plan.

This manual will inform you about all aspects of use and application of suited cleaning and disinfection products.

Oil, fats, ink, blood, urine and organic acids must be cleaned off immediately using a damp microfiber cloth and either lukewarm water or a mild washing-up liquid in warm water. Do not use any solvents, chlorides, polish (including wax polish) or chemical cleaning agents. Discoloration due to denim jeans or other clothing is not covered by any manufacturer warranty.

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The highest recommended concentration was tested in 50 applications each. The instructions for use of the detergents must be observed in every case. An impairment of the fabric in long-term use is not excluded. The change of cleaners and disinfectants should be avoided especially when using over a longer period, otherwise a degradation to the materials' properties can not be ruled out. Long-term use of alcohol-based cleaners or other solvents may negatively affect the coated surface and promotes embrittlement of the cover. As we have no control over the ingredients and manufacture of the recommended disinfectants, we have to reject any guarantee on the disinfectant resistance of the GREINER product.

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Lacquers, strong cleaners, detergents, xylene-based solutions, acetone, or MEK cause immediate damage and contribute to the deterioration of the material. The use of such cleaners is at the owner's risk. In order to check a not specified disinfectant on how well its impact is to the fabric a test at an invisible place is recommended. GREINER will not assume responsibility for discolouration caused by external contaminants and possible permanent staining caused by this phenomenon.

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The best care right from the start.

*After disinfection and cleaning of our products, proper care is particularly important to maintain their longevity and aesthetic appeal. For this purpose, we recommend the use of our **CARISMA cleaning & care lotion** as well as our **special care set** with the reusable **high-pile cloth**.*

In this way, you not only support the value retention of your products, but also ensure a pleasant and hygienic environment.

The result: your upholstery retains shine and glow despite heavy use.

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GREINER GmbH